F90 Uniprox Multiaxial Carbon-Fuß F90 Uniprox Multiaxial Carbon-Foot

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Sold By:

HI-TECH
Human Intelligence Technology, LLC

5611 S Kimbrough Ave, Springfield MO 65810
O: 888-750-6840
F: 573-207-4199
sales@humanitechnology.com | www.humanitechnology.com

Uniprox GmbH & Co. KG Tel. +49 (0) 36628-66-33 00
Heinrich-Heine-Straße 4 Fax +49 (0) 36628-66-33 55
07937 Zeulenroda-Triebes E-Mail info@uniprox.de

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8. Notes on reuse

The F90 Uniprox Multiaxial Foot is designed for single person usage and cannot be reused for other persons. According to the general Uniprox terms and conditions, this product is supplied on "Single patient use only" basis to be only used for the initial/original selected patient. This to protect the service provider, practitioner and the patient against potentially adverse consequences of infectious disease transmission, material malfunction in adapting to the configuration of the initial/original selected patient and/or reduced function of the device. Warranty expires when the product will be reused for another patient. As with all Uniprox products, this product must be prescribed and applied by a qualified practitioner to achieve maximum benefit of the product.

9. Information on the material used

The foot consists of plastic segments with a carbon footplate, an aluminum ankle adapter with a steel pyramid and elastomer bumpers.

10. Warranty and guarantee

Warranty is according to the sales terms and conditions of Uniprox GmbH & Co. KG. and only valid for design-, material-, or production related defects.
Warranty period F90 Uniprox Multiaxial Foot: 12 months.

For further information, please contact:

Human Intelligence Technology, LLC
O: 888-750-6840
F: 573-207-4199
sales@humanitechnology.com | www.humanitechnology.com

Order No.	Description	Features	Article No.
E-F80-OR	O-Ring	Color: black	4 602 029 00 00 001
E-F80-SS	Spectra-Sock F90		4 602 029 00 39 000
MM202	Lithium grease	Color: white	4 690 240 00 89 100

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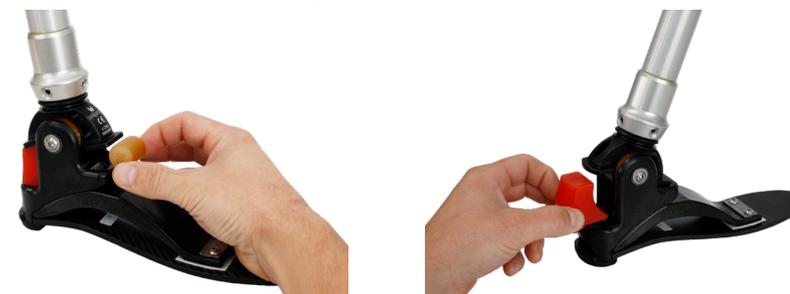
For changing the axial bumpers, first remove one of the axle bolts. Then fully slide out the axle. Plug the axial bumpers into the ankle part.

Secure the ankle bolts with Loctite® 243 after assembly. When using new bumpers, grease all slide and contact surfaces with Lithium grease® MM202.

5. Cleaning and disinfecting information

The F90 Uniprox Multiaxial Carbon Foot can be cleaned with standard soap and a damp cloth at >30°C. Do not use aggressive cleaners, as these may cause negative impact on the components of the foot.

6. Maintenance of warranty

Regular foot function checks are needed to guarantee the warranty of the F90 foot. Preferably every 6 months (maximum every 12 months). When a malfunction of the foot occurs that cannot be resolved by changing worn out parts, the foot needs to be returned to Uniprox for inspection. This to guarantee warranty.

7. Accessories and Spare Parts

The following parts are generally available. For other spare parts, please contact Uniprox customer service.

Order No.	Description	Features	Article-No.
E-F80-PV2	Bumper anterior 85 Shore	Color: beige	4 602 029 00 02 000
E-F80-PV3	Bumper anterior 95 Shore	Color: beige	4 602 029 00 03 000
E-F80-PV4	Bumper anterior 105 Shore	Color: black	4 602 029 00 04 000
E-F80-PM1	Bumper axial 85 Shore	Color: beige	4 602 029 00 05 000
E-F80-PM	Bumper axial 95 Shore	Color: beige	4 602 029 00 06 000
E-F80-PH1	Bumper posterior 55 Shore	Color: orange	4 602 029 00 16 000
E-F80-PH2	Bumper posterior 65 Shore	Color: beige	4 602 029 00 17 000
E-F80-PH3	Bumper posterior 75 Shore	Color: blue	4 602 029 00 18 000
E-F80-PH4	Bumper posterior 85 Shore	Color: red	4 602 029 00 19 000

F90 Selection of bumpers (shore hardness)

Weight Activity Level	41-68 Kg			69 - 76 Kg			77 - 95 Kg			96 -112 Kg			113 - 125 Kg		
	Front	Center	Back	Front	Center	Back	Front	Center	Back	Front	Center	Back	Front	Center	Back
Activity Level 2	85	85	55	95	95	65	95	95	75	95	95	75	95	95	75
Activity Level 3	85	85	65	95	95	75	95	95	75	95	95	75	95	95	85
Activity Level 4	85	85	65	95	95	75	95	95	75	105	95	85	105	95	85

Bumper Front	Bumper Center	Bumper Back
105 Shore A	95 Shore A	85 Shore A
95 Shore A	85 Shore A	75 Shore A
85 Shore A		65 Shore A
		55 Shore A

4. Fitting instructions

For the correct A/P position of the foot, position the socket's reference line (socket – ankle) approximately 0-5 mm posterior of the socket position.

The F90 foot will be supplied pre-assembled. For preferred gait and comfort, correct bumper selection is needed. To choose the desired setting, start with the stiffest bumper and change to softer bumpers when necessary. See the selection matrix in point 3.

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1. Intended use

The F90 Uniprox Multiaxial Carbon Foot is designed for Below Knee and Above Knee prosthesis. Patient Mobility levels 2, 3 and 4 (according MDS) and available in foot sizes 24 – 29 cm. Patient body weight limit is 125 kg. Additional loads must be taken into account when calculating the weight.

2. Application information, safety instructions and warning notices

- The foot can be used at temperatures from -10°C to +50°C.
- The patient should stop using the foot and immediately contact his/her practitioner when the foot shows changes or faults in rollover behavior, generates noise or impacts from external forces. When the practitioner cannot solve the occurred faults, he/she should contact Uniprox customer service.
- Do not use aggressive cleaners to clean the prosthetic foot.
- To select the most optimal and safe foot for the patient, please use the selection matrix (see point 3).
- The product must not be changed or applied in any improper way. Non-compliance can impair the function of the product and thus no product warranty can be honored.
- The safety of the patient/user is not guaranteed if the product is not used according intended purpose. Incorrect usage will void warranty.

When all usage directions are observed as informed, no additional risks are expected. Momentarily no contraindications are known.

3. Technical data

Order No.	Available	Heel	Installation	Weight	Article No.
Size/Side/	Sizes	Height (A)	Height (E)	in size 27	
F90- <input type="checkbox"/> <input type="checkbox"/>	24-29	10 mm	135 mm	810 g	4 603 030 0X 00 0ZZ

Side: L = left, R = right
 Example: Foot F90 in size 27, Side left
 Order-No.: F90-27L

X = Side: 1 = left, 2 = right
 ZZ = Size (See available size)
 Example: Foot F90, Side left, Size 27,
 Article No.: 4 603 030 01 00 027

Sold By:

HI-TECH
 Human Intelligence Technology, LLC

5611 S Kimbrough Ave, Springfield MO 65810
 O: 888-750-6840
 F: 573-207-4199

sales@humanitechnology.com | www.humanitechnology.com